KILYA® patch

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KILYA® patch

Kilya® patch is a medical device.

Kilya® patch is a heating patch that is used to relieve painful periods, also known as dysmenorrhoea.

Dysmenorrhoea is the medical term for menstrual cramps or pain. Most menstruating women have some cramps in their lower abdomen, sometimes extending to the lumbar area, hips and legs. For some, this is just annoying, while for others it is serious and disabling.

What are the symptoms of dysmenorrhoea? Dysmenorrhoea is usually characterised by a menstrual cramp that starts between several hours before and a few hours after the onset of menstrual bleeding. Dysmenorrhoea may be accompanied by vomiting, fatigue, back pain, headaches, dizziness and diarrhoea. Primary dysmenorrhoea is due to the excessive release of prostaglandins. Prostaglandins are hormone-like chemicals that are released during the menstrual period. They are the cause of pain. They increase the amplitude and frequency of uterine contractions and cause vasoconstriction and spasm of the uterine arterioles, resulting in cyclic cramps of the lower abdomen.

HOW DOES THE KILYA® PATCH WORK ?

The working principle of the Kilya® patch is based on thermotherapy. The heat of the patch is generated by rapid oxidation of iron powder associated with quartz monzonite powder.

The Kilya® patch warms and relieves muscle tension in the lower abdomen.

The body's natural circulation diffuses heat and increases blood flow to the uterus. The constant soothing heat activates heat receptors in painful areas and blocks the detection of pain signals.

Most painful menstrual cramps are caused by strong muscle contraction. The heat treatment of the Kilya® patch helps to relax the muscles and thus effectively relieves menstrual cramps, as well as reducing and effectively relieving pain.

Be sure to fix the Kilya® patch to your underwear. Do not apply the patch directly to the skin!

FEATURES


- Can be used for up to 8 hours
- Maximum temperature reached: 50°-60°
- Simple and comfortable to use, it is designed to be portable and easy to use by applying the patch to the concerned area (Hot Pack type sticker)

HOW TO USE THE KILYA® PATCH

1. Take the Kilya® patch out of the sachet and shake it gently: the heating reaction will start automatically and progressively

2. Remove the film in order to reveal the self-adhesive area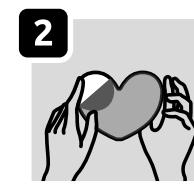

3. Fix the Kilya® patch to your underwear: panties, bodysuit, etc.

WARNING: The Kilya® patch should not be placed directly on the skin.

PRECAUTIONS FOR USE

- Do not apply directly to the skin to avoid any risk of burns.
- People with sensitive skin should take particular care.
- To be used under supervision with children and people with disabilities.
- For external use only.
- Don't use in Pregnancy
- Use of adolescents from 12 years of age
- Should not apply the patch with abnormal menstrual cycles

The distributor cannot be held responsible if the precautions for use are not respected.

STORAGE

- • Store in a well-ventilated place; avoid exposure to sunlight.
- Carefully handle sachets containing patches; avoid shocks or damage.
- Keep out of the reach of children
- • Do not use if package is damaged
- • Do not re-use

PACKAGING

Box of 3 heating patches.

COMPOSITION

Activated carbon, quartz monzonite powder, iron, sodium acrylate, vermiculite, salt.

Available exclusively from your pharmacist.

The Kilya® patch is registered as a medical device in Belgium and the Grand Duchy of Luxembourg

0477

Manufacturer:

Wooshin Labottach Co., Ltd.
B-109, 915-1020, Songdo BRC Smart Valley
Songdo-dong, 30, Songdomirae-Ro
Yeonsu-gu, Incheon, Korea

EC REP Authorized representative:

Wooshin Lapache d.o.o II Industrijska cesta 1J, 1290 Grosuplje, Slovenia II

Distributor:

Cole Pharmaceuticals SRL
21, champ de Maubroux, 1390 Grez-Doiceau
Belgium
www.colepharma.com

colepharmaceuticals